


## 4 Piece Orthopedic Wedge Pillow Set



### An Ergonomic Design

- The zippered cover is made of a soft, breathable, machine washable material that is easily removable
- Help reduce snoring, improves circulation and relieves pressure on sensitive areas. It can also be used to comfortably sit up and watch TV or read
- Can be used anywhere—in bed, on the floor, or the couch
- It can be used after surgeries, accidents and injuries
- To aid symptoms of indigestion and help reduce stress on the back
- Helps alleviate symptoms of respiratory conditions
- Keeps legs in stress-free posture and helps increase leg circulation

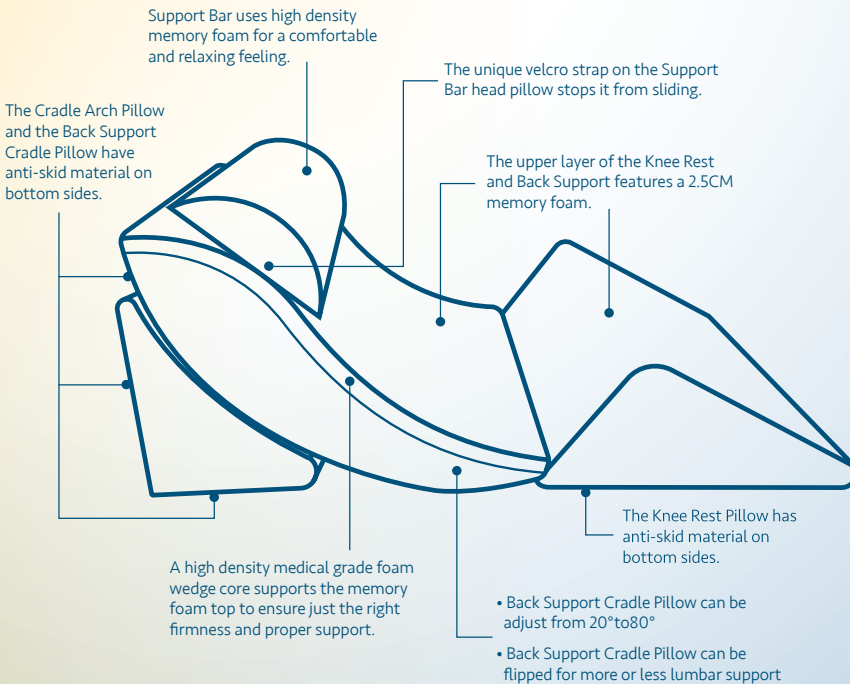

Cradle Arch



Support Bar



Back Support Cradle



Knee Rest

**Care Instructions:** Covers can be removed and machine washed on delicate cycle. Maximum temperature 60°C (140°F). Do not bleach. Do not iron. Do not dry clean. Tumble dry low or air dry. Foam cores can be spot cleaned with a moist, soapy rag. Let air dry. Foam cores may never be placed in the washing machine.

**Important Notice:** The 4 Piece Orthopedic Wedge Pillow Set is shipped in compressed packaging. Remove Cushions from plastic packaging to begin decompression. The process will begin immediately. Allow several hours for cushions to decompress before use.

**Please Note:** Many factors contribute to the speed of your cushion's decompression. In the vast majority of cases your cushions will decompress in the time frame indicated. In the rare case that it does not, please allow the cushions to decompress overnight.

**Please Also Note:** Although you may use the cushions at this point, it may take up to 24-48 hours for both cushions to reach its peak firmness.

New memory foam has a slight scent. Airing out for 24 to 48 hours will help dissipate the scent.